TAPE CORRECTION LOOP

TYP _____

DAUMEN

THUMB/POUCE/
POLLICE/POLEGAR
KCIUKI/PULGAR

KORREKTUR-TAPE
CORRECTION TAPE
BANDE CORRECTRICE
TAPE CORRETTIVO
FITA DE CORREÇÃO
TAŚMA KORYGUJAÇA
CINTA CORRECTORA

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The TAPE LOOP is a non-adhesive, skin-friendly tape. It limits the rotation possibilities of the thumb without any side effects and supplements conservative therapy options. However, the mobility of the thumb is basically maintained. It can be used in the long term for stabilisation, support and correction of problems caused by overstraining and functional causes in the thumb joint area. Pain can be relieved if necessary. The thumb loop can also be used preventively and postoperatively. The TAPE LOOP is suitable for everyday use – it can be used in gloves and wet environments.

The correct application of the TAPE LOOP

Place the TAPE LOOP with the rubberised side on the skin. Pass the thumb through the loop and position the loop on the inside of the thumb. The widening of the loop lies against the inner ball of the thumb (fig. 1).

The crossing of the loop is on the outside of the hand (fig. 2).

Pull the Velcro through the metal loop and tighten the loop sufficiently and press the Velcro on (fig. 3). Try to spread your thumb as far as possible before tightening it. Tighten the loop only as much as you feel comfortable. However, use enough pull so that the loop can unfold its effect.

fig. 1

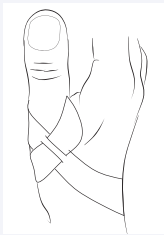
fig. 2

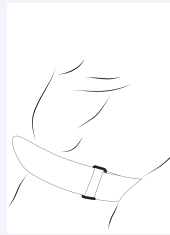
fig. 3

Wearing time

Please wear the TAPE LOOP only for about one hour in the beginning and increase it over the next few days as you are able. The joint and the surrounding structures need some time to get used to the new position. When working with the hand, the loop should support you in maintaining the stretched hand position as well as possible.

Care instructions


Use the COMPRESSANA laundry net · do not use fabric softener

Exercises

1. Thumb spreader (mobilisation)

Grasp the opposite ball of the thumb with the fingers and work the muscles of the thumb outwards with finger pressure.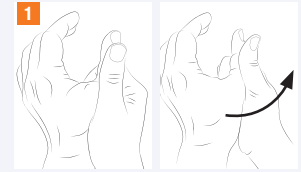

2. Ball hand (activation)

Lay your hand flat, spread it as wide as possible, then curl it from the thumb and little finger side as if you want to grasp a large ball. Do not grip!

Risks and side effects

Follow the instructions of the medical personnel. If pain or negative changes occur during use, refrain from wearing TAPE LOOPS and contact your doctor immediately. If you have any intolerance to materials or ingredients, please consult your doctor, pharmacist or specialist retailer before use.

The manufacturer is not liable for damage and injuries due to improper handling or non-observance of the instructions, as well as in case of incompatibilities to materials/ingredients.

Storage / Durability / Warranty

Store at room temperature and in a dry place. Avoid heat and humidity. Protect from sunlight.

The warranty period for the COMPRESSANA TAPE LOOP is 6 months. The taping loop is subject to application and material-related aging and to natural wear and tear during use.

In case of item damage, please contact your specialist retailer or dispensing chemist. Do not repair the TAPE LOOP yourself as this could affect the therapeutical effect.

Every hand is individual

Choosing the right size and a perfect fit are crucial. You will find the size or size information stamped into the tapes. In case of doubt or if you have any problems, please contact your specialist dealer and have the fit examined.

For hygienic reasons, this product is intended for one person only.

Disposal

There are no special criteria for disposal. COMPRESSANA TAPE LOOPS can be disposed of with household waste, the packaging material with waste paper recycling.

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COMPRESSANA GmbH